



Technical Information Clorox Healthcare® Fuzion™ Cleaner Disinfectant



PRODUCTS
& SAFETY

DISINFECTION DIRECTIONS FOR USE#



1. REMOVE gross soil if visible. For *C. difficile* spores or TB, always clean surface prior to disinfecting.†
2. (If trigger is in locked position) SQUEEZE the red tab under the nozzle and push downward to unlock the trigger.
3. SPRAY 6"-8" from surface until surface is completely wet.
4. To DISINFECT, let stand for 1 MINUTE. To kill *C. difficile* spores, allow 2 minutes contact. Then WIPE. Allow to air dry.

Special label instructions for cleaning prior to disinfection against *Clostridium difficile* spores:

- **Personal Protection:** Wear appropriate barrier protection such as gloves, gowns, masks and eye covering.
- **Cleaning Procedure:** Fecal matter/waste must be thoroughly cleaned from surfaces/objects before disinfection by application with a clean cloth, mop and/or sponge saturated with the sporicidal product. Cleaning is to include vigorous wiping and/or scrubbing, until all visible soil is removed. Special attention is needed for high-touch surfaces. Surfaces in patient rooms are to be cleaned in an appropriate manner, such as from right to left or left to right, on horizontal surfaces, and top to bottom, on vertical surfaces, to minimize spreading of the spores. Restrooms are to be cleaned last. Do not reuse soiled cloths.
- **Infectious Materials Disposal:** Materials used in the cleaning process that may contain feces/wastes are to be disposed of immediately in accordance with local regulations for infectious materials disposal.
- **To Disinfect:** Spray 6–8 inches from surface until surface is thoroughly wet. Allow this product to remain wet for 2 minutes. Then wipe.

Special instructions for using this product to clean and decontaminate against HIV-1 and HBV on surfaces/objects soiled with blood/body fluids:

- **Personal Protection:** When handling items soiled with blood or body fluids, use disposable impervious gloves, gowns, masks and eye coverings.
- **Cleaning Procedure:** Blood and other body fluids must be thoroughly cleaned from surfaces and other objects before applying this product.
- **Contact Time:** Allow surface to remain wet for HBV and HIV-1 for 1 min.
- **Disposal of Infectious Materials:** Use disposable impervious gloves, gowns, masks and eye coverings. Blood and other body fluids must be autoclaved and disposed of according to local regulations for infectious waste disposal.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes, but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection.

See label for special instructions for cleaning prior to disinfection against *C. difficile* spores and TB. Look on the product label for additional directions for other applications such as for food-contact surfaces, veterinary sites and instrument precleaning or disinfection.

† This is a one-step product. Precleaning is required only if blood, vomit, fecal matter or other soil is visible, or in advance of disinfecting *C. difficile* spores or TB. Use product as directed.